


Quick Start Guide


This guide is designed to help you set up Orka One and navigate your way through Orka Health App.

1 Setup

① Check your hearing aids

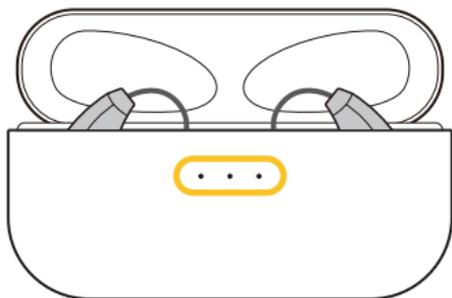


- The LED light indicates charging case power. Open the lid, three lights will turn on. This means your case is fully charged

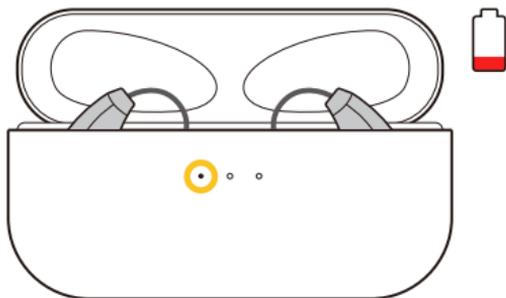


- If only one light is on, please charge your case before proceeding with setup

② Download Orka Health app

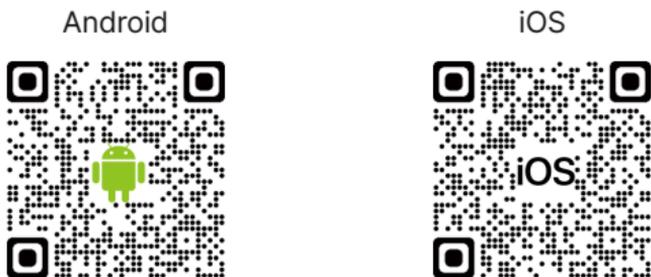


- Before connecting to Bluetooth, scan QR code and download the app

③ Login

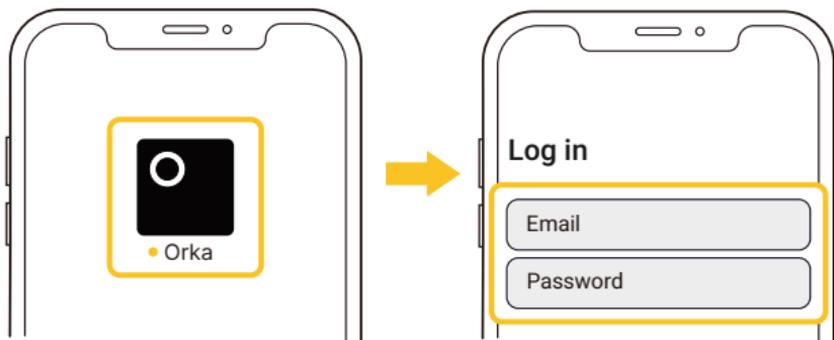

- Use the account you created at checkout
- Or create a new account to login


④ Follow app instructions

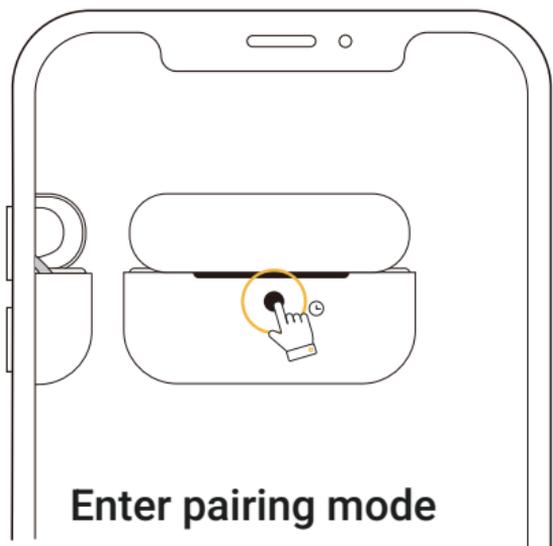

- **Questions?** Get support through support@hiorka.com
- Or call us at [\(800\)653-6022](tel:8006536022)


2 Basic Info

① Use hearing aids

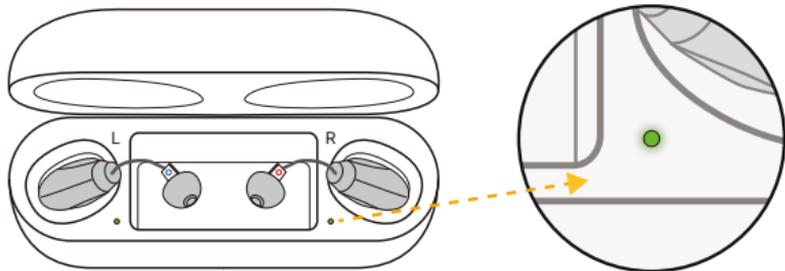


- Open the lid, the LED will light up to indicate your hearing aids are turned on

② Distinguish left & right

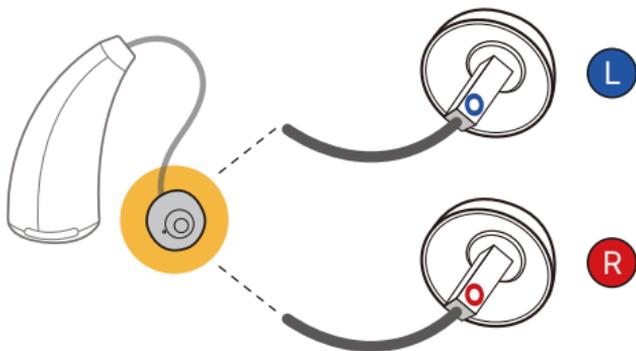

③ Select proper domes

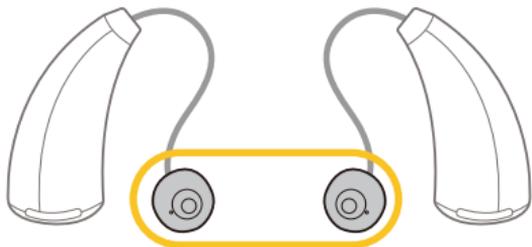

- There are 3 different types of domes in size: Small, Medium and Large



- **Open dome:**
for more natural sound



- **Closed dome:**
for better bass response




- **Double dome:**
for greater sound enhancement


- Your hearing aid comes with Closed Medium dome already attached

3 Charging

① Place each hearing aid in the case

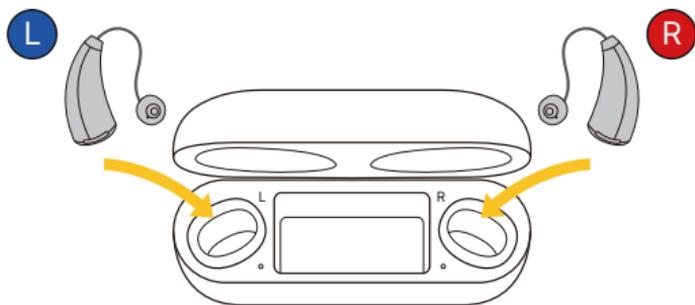


• **Close** the lid to start charging

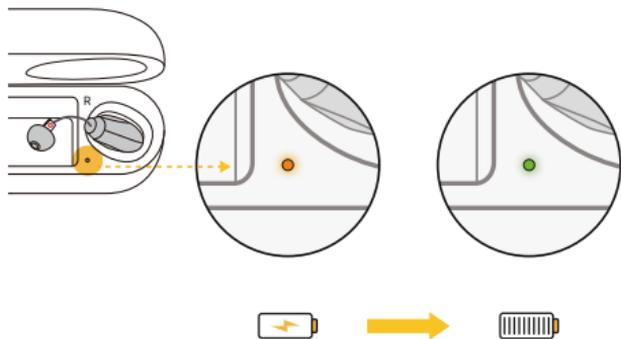


• A full charge will take about **1.5** hours

② Charge the case

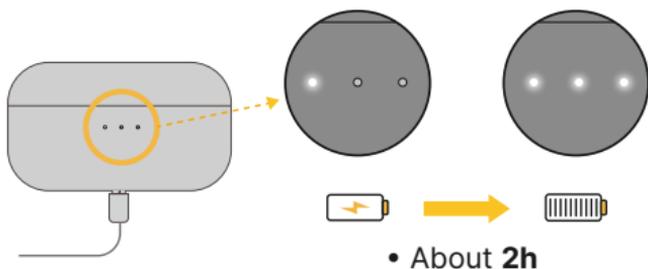

③ Low battery indication

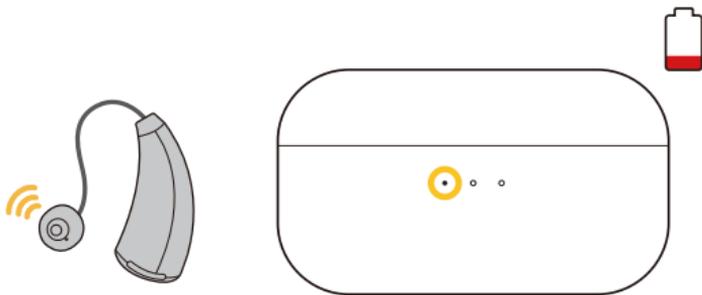

- Charge hearing aids when you hear a "**low battery**" message
- Charge the charging case when there is only one light on

4 Two Modes

① Hearing aid mode



- For sound enhancement

② Bluetooth mode




- For streaming and calls

③ Switch modes




- Double tap the hearing aid to answer phone calls and switch between different modes

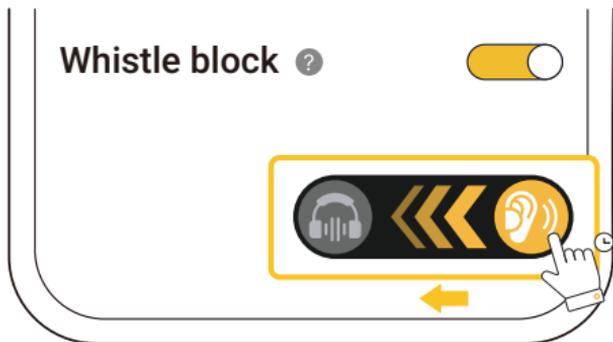

- You can also press and slide to switch mode in the Orka Health app


5 Hearing Aid Controls

① DeNoise mode

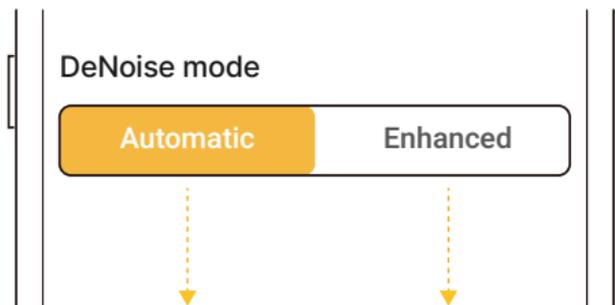

- For common settings
- For noisy settings

② Adjust volume

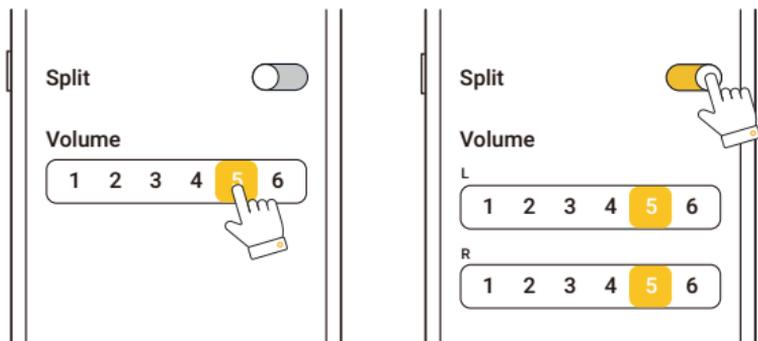


- Adjust the volume of both ears at the same time
- Turn on "**Split**" to adjust the volume separately

③ Hearing profile




- Access "**Menu**" in the top left corner

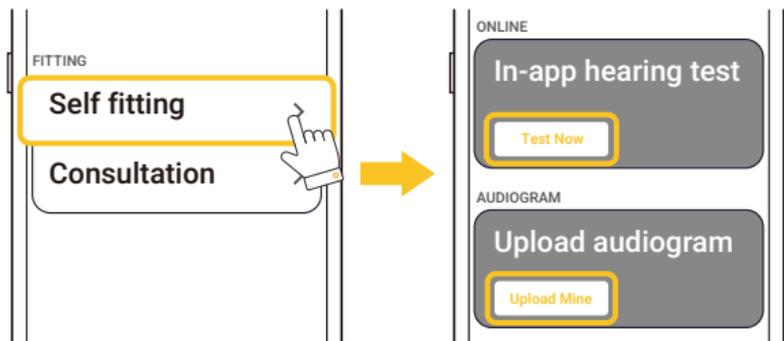


- Upload your audiogram if you have one available
- Take the in-app hearing test if you do not have one with you

6 Remote Care

① Remote fitting

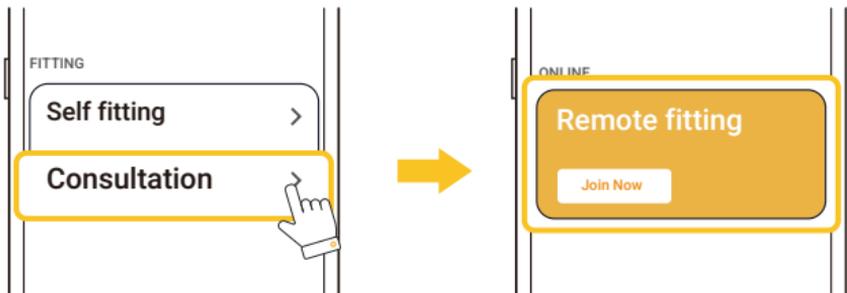


- Enter Menu in the top left corner. Find "**Remote Fitting**" under "**Consultation**"

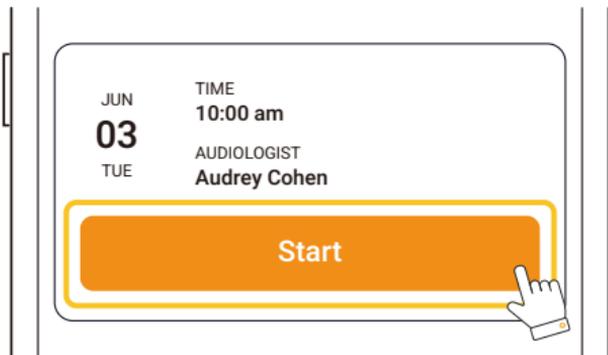


- Schedule an appointment to start a fitting session with an audiologist

② Contact us

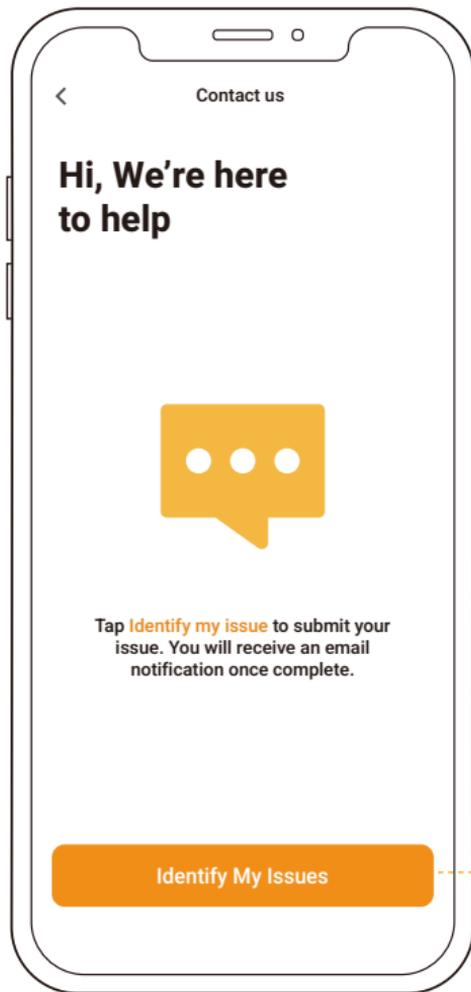

- Got questions or issues? Contact us through **"Menu" → "Contact Us"**

- Report your issues here

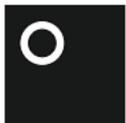

ORKA