

TAKE A HEARING TEST WITH US



1 About in-app hearing tests

In-app hearing test allows you to personalize Orka One to your unique hearing profile. We recommend taking the in-app hearing test if you don't have an audiogram.

Please Note: The in-app hearing test is not intended to replace professional assessments by audiologists. But it can support your hearing loss level.

You can go to **App>>Menu>>Self-fitting>>More>>Preset** for a preliminary setup. However, presets are basic adjustments and not tailored to give you a personalized experience. We recommend taking the in-app hearing test when you have time.

2 How an in-app hearing test works

- Please make sure you are in a **quiet** place.
 - ▮ The microphone on your smartphone will detect if your environmental noise is too high to complete the test accurately.
- Use **Double domes** (enclosed in the envelope), as they have a better sound-proof effect.
- Please make sure the domes fit properly and snugly in your ear canal.
- Please stay focused while taking the test, tap "I hear the beep" when you hear the sound. (If you do not hear any, be patient and wait until you hear the tone).

3 Things that may influence your results

- Not using double domes.
- Domes not fitted properly during the test.
- High level of background noise.
- Lack of attention.

4 What comes after the hearing test

After the test is complete, you will be able to view your result. Orka Health App will generate a new hearing profile automatically each time you take the hearing test. Make sure you take some time to get accustomed to the new hearing profile. You can take the test as many times as you would like, but remember that you can always book a fine-tuning session with us via **App>>Menu>>Consultation>>Remote fitting**.